

Adjustable Overbed Table, With Castors - 90291

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Width	410mm
Length	680mm
Height floor to table top	730mm - 1000mm
Table Top	600mm x 380mm
Footprint	560mm x 350mm
Maximum Weight Limit	5kg (11lb)

Use of the Product

This product is designed to be used over a bed or chair to enable the user to support lightweight materials such as newspapers or meals.

Assembly Instructions

1. Carefully unpack the overbed adjustable table and examine all parts for any damaged or loose fittings
2. Screw the four castors into the holes in the underside of the frame base
3. Place each leg over the base spigots
4. Insert the E-clips to attach the legs to the spigots
5. Slide the chrome table supports into the fixing saddles on the underside of the table. Ensure the spring pin locates in the centre of the fixing saddle to prevent movement
6. Insert the table support frame into the uprights and tighten the hand screws. The table angle and height can be adjusted at this time

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050

Adjusting the Product

To adjust the height and / or angle of the table to the most comfortable position, first slacken the two hand screws by turning anticlockwise. Adjust the height and angle and then re-secure by turning the hand screws clockwise.

Cleaning

To clean the table, use a soft and slightly moistened cloth with a mild detergent. All parts of the overbed table can be cleaned in this way. It is not recommended to use any bleach, very hot water or abrasive surface cleaners as these may result in damage to the top or frame.

Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands

UK | CE
CA

UK | UK
CA | NI

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF
E: customerservices@integrity-healthcare.co.uk
T:+44(0)800 043 0050