

## Toilet Frame, Height and Width Adjustable - 90438

### Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	680 – 740mm
Overall Depth	550mm
Overall Height	740 – 860mm
Width Between Arms	575 – 650mm
<b>Maximum User Weight</b>	<b>190kg (30st)</b>

### Use of the Product

This product has been designed to provide support in toileting to users who require help in standing or sitting. The frame is made from a rigid double strut steel tube that locates around the toilet. There are moulded plastic arms attached to provide a comfortable grip and easy cleaning.

### Assembly

1. Unpack and check that all parts are complete
2. Fit the horizontal bars to join the two side frames using the E-clips provided
3. Insert the extending legs into the frame using the E-clips, ensuring the clips are fully inserted and fit snugly to the frame

### Adjusting the Product

The height of the frame can be adjusted by removing the E-clips on each of the legs and lifting (or lowering) the frame to the desired height. Once the desired height has been achieved, replace the tube E-clips in the nearest hole in both tubes locking the frame in position. The width can be adjusted by removing the E-clips on the horizontal bars and moving the frame to the desired width. Replace the E-clips in the nearest holes ensuring they pass through both bars to secure them.

## Cleaning

Use a non-abrasive, mild detergent in warm water and a soft cloth to clean the toilet frame. Rinse with clean warm water and dry with a soft cloth. **DO NOT** use any form of abrasive cleaner as it could cause damage to the product.

**Thank you for choosing an Integrity product.**

EUCEREP B.V.  
Roald Dahllaan 33 , 5629MC  
Eindhoven  
Netherlands

UK | CE  
CA

UK | UK  
CA | NI

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park  
Cowbridge, Vale of Glamorgan, CF71 7PF  
E: [customerservices@integrity-healthcare.co.uk](mailto:customerservices@integrity-healthcare.co.uk)  
T:+44(0)800 043 0050