

Perching Stool, HF Seat and Back with Steel Arms - 90245

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	550 – 630mm
Overall Depth	410 – 450mm
Seat Width	330mm
Seat Depth	265mm
Seat Height	535 – 680mm
Width Between Arms	455mm
Height of Arms from Seat	175mm
Maximum User Weight	165kg (26st)

Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If you spot any of these issues, please contact your equipment provider

Use of the Product

This product has been designed to provide support in to users who require help to remain standing for more extended periods of time. The legs are designed to longer at the rear of the product to best support a user in daily tasks.

When moving always lift the perching stool by placing your hands on the tubular metal frame beneath the seat.

Do not allow children or unauthorized persons to play with the equipment or use without proper supervision.

Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury.

Do not use the perching stool outside the home or on uneven surfaces (such as in the

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050

garden) as this will cause stress to the legs and possible damage which may cause injury.

Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury.

Adjusting the Product

To adjust the height, remove the E-clip from the extension leg and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg. Note the perching stool is designed to slant to the front. Do not adjust any of the legs to make the seat level as this will cause excessive stress to the legs compromising user safety and possibly leading to permanent damage of the perching stool, including fracture of the tubular and other sections.

Once you have achieved the desired height reinstall the E-clip, ensuring that it has passed through the extension leg and that it fits neatly and snugly around the outside leg.

Cleaning

Prior to cleaning the chair, please remove all E-clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

To clean the product, please use a mild detergent with warm water and a soft cloth to wipe down the framework on at least a weekly basis or more frequently if needed. If required, disinfect with a chloride disinfectant, ensuring thorough rinsing and drying prior to reuse.

Maintenance

It is advised to periodically check the frame for signs of wear and tear, if any problems are noted please contact your equipment provider.

Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF
E: customerservices@integrity-healthcare.co.uk
T:+44(0)800 043 0050