

Integrity Static Shower Chair - 90092

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	530mm
Overall Depth	485 - 510mm
Seat Width	415mm
Seat Depth	380mm
Seat Height	480 - 580mm
Width Between Arms	480mm
Maximum User Weight	165kg (26st)

Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, cracked or damaged seat or bent tube
- Ensure the seat is securely fitted into the frame prior to being used

Use of the Product

This product has been designed specifically for use in a shower. It comes complete with a heavy duty plastic seat with perforations to allow water to flow through easily. There are arms provided to provide additional support. This chair is designed as a temporary seating aid and should not be used as a step. This chair is suitable for use in showers, bathrooms & wet rooms. It is **not** suitable for use in swimming pools.

Adjusting the Product

To adjust the height, remove the E-clip from the extension leg(s) and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050

Once you have achieved the desired height, reinstall the E-clips, ensuring they have passed through the extension leg and that they fit neatly and snugly around the outside leg.

Cleaning and Maintenance

Prior to cleaning the shower chair remove all E-clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the plastic seat and metal parts using a non-abrasive soft cloth with a mild detergent. This should be carried out weekly or more frequently if soap build-up is visible. Abrasive cleaners and/or abrasive cleaning pads could seriously damage the seat and void the warranty. If required, disinfect with a chloride based disinfectant ensuring thorough rinsing and drying prior to further use.

Periodically check to ensure all components are present, correctly assembled and suitably fitted.

Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050