

Commode, Fixed Arm and Height Adjustable - 90096

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	540mm
Overall Height	810 – 900mm
Overall Depth	510 – 560mm
Seat Width	400mm
Seat Depth	400mm
Seat Height	480 – 580mm
Width Between Arms	490mm
Maximum User Weight	160kg (25st)

Assembly Instructions

- Carefully unpack and examine the product to ensure it is complete.
- Before use, check for any signs of damage e.g. cracked weld, flaking of paint, bent tube. If you spot any of these issues, please contact your equipment provider.

Use of the Product

This product has been designed specifically to aid toileting requirements and consists of a coated tubular frame with a cushioned over seat and back rest, covered in a flame retardant, waterproof vinyl covering as well as a plastic hinged toilet seat and 5 litre pan and lid.

Adjusting the Product

- Ensure the legs of the commode are all set to the same height using the E-Clips
- Please empty the potty at regular intervals and be aware of its maximum capacity of 5 litres
- Do not attempt to move the commode whilst occupied or in use
- To locate the potty in the commode, raise the toilet seat to a vertical position and place the potty on the support brackets ensuring an accurate location of the wide brim onto the brackets
- Remove the lid (potty only) fold the potty handle down and lower the toilet seat into

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050

the horizontal position

- Once the commode has been used, repeat the process in the reverse order ensuring that the lid (potty only) is correctly placed to avoid possible spillage

Cleaning

Prior to cleaning the chair, please remove all E-clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

To clean the product, please use a mild detergent with warm water and a soft cloth to wipe down the framework on at least a weekly basis or more frequently if needed. If required, disinfect with a chloride disinfectant, ensuring thorough rinsing and drying prior to reuse.

Maintenance

It is advised to periodically check the frame for signs of wear and tear, if any problems are noted please contact your equipment provider.

Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050