

## Aluminium Shower Commode Chair – 90209

### Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	460mm
Overall Depth	860mm
Under Seat Height	485mm
Aperture	180 x 280mm
Seat Height	510mm – 560mm
Width Between the Arms	440mm
<b>Maximum User Weight</b>	<b>160kg (25st)</b>

### Use of the Product

The product is designed to be used as an aid for showering or toileting. It comes with a potty and lid or can be wheeled over most conventional and disabled toilets. Please note:

- Only use on level ground
- When stationary apply the brakes on the wheels and ensure the brakes are released before the chair is moved again
- Always ensure that the device is stationary and brakes applied when transferring to and from the device. Swing away the leg rests before getting in or out of the device. The user should always remove their feet from the leg rests before swinging the leg rests away.
- Do not lean out of the device
- Ensure that all castors are set to the same height and that there is not a significant tilt on the seat
- Be careful of pinch points when rising from or lowering onto the seat, or when you are removing the potty

## Assembly and Adjustment Instructions

Check that all components are secure before use.



1. Take the main frame and secure the central bar using the butterfly knobs



2. Insert the height adjustable castors and secure using the star knobs

3. Insert the backrest and fix using the larger star knobs as shown

4. Locate the seat by sliding the back of the seat into the frame. Then press down on the whole of the seat until secure. Add the potty and aperture cover if required (see instructions)



### Footrests Function and Removal

1. The footrests insert into the frame upright, the plastic keeper presses around the upright tube to lock the footrest to the frame and then locks with the spring pin as shown
2. The footrests can swing in and out if the user requires this by pulling the lever upwards. Flip up the footplates prior to swinging the footplates away. When returning the footrests to the front ensure that the spring pin is fully engaged to lock the footrests into position
3. To remove the footrests, lift up the footplates, swing out the footrest and lift up and out of the frame



### Footplate Adjustment

1. To adjust the height of the footplate, unscrew the small star knob and remove the bolt. Slide the footrest to the required height and secure the footrest by returning the bolt and tightening
2. To adjust the footplate angle, slacken the bolt near the bottom of the inner tube (chrome) using an Allen key. Adjust the footplate to the required angle and re-secure the bolt.

### Castor Operation

The castors can be locked so that the wheel cannot move forwards or reverse nor will the castor fork swivel. To lock the castor press down on the locking tab with a foot (as shown). To release press the other side of the locking tab with a foot.



### Castor Adjustment

The height of the chair and seat can be altered by adjusting the castor height. To adjust the castor height, loosen the star knob and remove. Reposition the castor stem to the required position and reattach the E-clip or nut and bolt. Repeat for the other 3 castors ensuring they are all set at the same height.



## Potty Installation and Removal



The potty is installed and removed by sliding the potty in to the two rails mounted to the underneath of the seat. To install, slide the potty underneath the seat from the rear of the chair, ensuring the potty sits on both rails of the seat. Do not install or remove the potty whilst in use to prevent any trapping risk.

Cover the potty with the lid to reduce the risk of spillages and carefully dispose of the potty contents in a sluice or WC. Do not dispose of in an open drain or sink as this can cause serious health risks.

The potty can be cleaned and disinfected manually or in an autoclave/aqueous decontamination machine. It should be cleaned after each use. Replacement potties are available from your supplier.

## Seat Operation and Removal

The seat is a horseshoe shape with removable cover for the aperture. When used as a shower chair, it is recommended that the aperture cover is removed to allow better access and drainage.

The seat clips on and off the frame. To remove the seat for cleaning, first remove the potty and aperture cover. Then lift up the front of the seat to release and slide the seat away from the chair. To attach the seat first locate the seat by sliding the back of the seat into the frame, then press down on the whole of the seat until secure.

## Arm Operation

The arms can flip up to aid side transfer. Simply lift the front of the arms to flip up, and then push down to lower. The brakes should always be applied before transferring.

## Cleaning and Maintenance

The chair should be stored in a dry environment, away from direct sunlight. When in storage the chair should be kept clean and dust free. This can be done with a duster or damp cloth.

The chair can be hosed down or wiped down with mild detergent. After washing, wipe the chair dry. It is recommended that all castors, joints and fasteners are sprayed with a water dispersing agent.

**Thank you for choosing an Integrity product.**

EUCEREP B.V.  
Roald Dahllaan 33 , 5629MC  
Eindhoven  
Netherlands



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park  
Cowbridge, Vale of Glamorgan, CF71 7PF

E: [customerservices@integrity-healthcare.co.uk](mailto:customerservices@integrity-healthcare.co.uk)

T:+44(0)800 043 0050