

## Integrity Shower Stool - 90115

### Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	510mm
Overall Depth	300mm
Seat Width	380mm
Seat Depth	265mm
Seat Height	490-600mm
Width Between Arms	465mm
<b>Maximum User Weight</b>	<b>160kg(25st)</b>

### Assembly Instructions

This product comes fully assembled.

### Use of the Product

- When moving always lift the shower stool by placing your hands on the tubular metal arms
- Prevent children and unauthorised persons to play with the equipment or use without proper supervision
- Do not rock back and forth or sideways when using the stool as this will cause excessive stress and possible damage and failure which may cause injury
- Do not use the shower stool outside the home or on uneven surfaces (such as in the garden) as this will cause stress to the legs and possible damage which may cause injury
- Do not place fingers or foreign objects into any of the exposed tubular sections as this can lead to serious injury

### Adjusting the Product

To adjust the height, remove the E-clips from the extension legs and extend all legs evenly to the desired height ensuring you do not exceed the last punched hole of the extension leg.

Once you have achieved the desired height reinstall the E-clips, ensuring that they have passed through the extension leg and that it fits neatly and snugly around the outside of the leg.

### **Cleaning and Maintenance**

Prior to cleaning the shower stool remove all E-clips and the four extension legs with the ferrules fitted. Clean these items separately to the main frame.

Clean the plastic seat and metal parts using a non-abrasive soft cloth with a mild detergent. This should be carried out weekly or more frequently if soap build-up is visible. Abrasive cleaners and/or abrasive cleaning pads could seriously damage the seat and void the warranty. If required, disinfect with a chloride based disinfectant ensuring thorough rinsing and drying prior to further use.

Periodically check to ensure all components are present, correctly assembled and suitably fitted.

**Thank you for choosing an Integrity product.**

EUCEREP B.V.  
Roald Dahlilaan 33 , 5629MC  
Eindhoven  
Netherlands



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park  
Cowbridge, Vale of Glamorgan, CF71 7PF  
E:[customerservices@integrity-healthcare.co.uk](mailto:customerservices@integrity-healthcare.co.uk)T:+44(0)800 043  
0050