

2 Shelf Dinner Trolley with Brakes, Height Adjustable - 90515

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Frame Height	680mm – 960mm
Width Between Handles	470mm
Tray Table Width	460mm
Tray Table Depth	340mm
Footprint	520mm x 550mm
Maximum User Weight	140kg (22st)

Use of the Product

This product has been designed to help the user to safely transfer items around the home. The trolley is not designed to be sat upon. Although this item is used when walking, **it should not be used as a walking aid.**

Assembly Instructions

1. Unpack and check that the parts are complete
2. Place the frame upside down exposing the fittings for the castors. Place the steering castors on the front, upright edge and then insert the rear castors with the brakes ensuring the wheels are on the outside. All wheels are secured with E-clips
3. Place the trolley on the castors and insert the handles ensuring that the handles correspond to the braked castor
4. Secure the handles using the star knobs and washers



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050

Adjusting the Product

Adjust the height of the product using the E-clips on the legs ensuring all the legs are at the same height and the E-clips are fully inserted.

Cleaning

Use a non-abrasive mild detergent along with warm water & a soft cloth to clean the frame. Rinse with clean warm water and a soft dry cloth. **DO NOT** use any form abrasive cleaner, cloth or sponge with this product as this could cause damage to the product.

The trays are designed to be a clip-on / off fit for cleaning purposes. To remove the trays, lift the front edge of the tray first and then push up from the underside. To fit the tray, push firmly down with the palm of a hand spread over the flat surface of the tray. Ensure that both trays are level with the framework.

Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands

UK | CE
CA

UK | UK
CA | NI

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF

E: customerservices@integrity-healthcare.co.uk

T:+44(0)800 043 0050