

## Glideabout Commode, 4 Braked Castors, Height Adjustable - 90675

### Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Seat Width	430mm
Seat Depth	430mm
Seat Pad Width	435mm
Seat Pad Depth	435mm
Seat Height	510mm
Depth (without footrests)	495mm
Footrest Depth	280mm
Height to Armrests	570mm
Height from Footrest to Seat	385 – 500mm
Height to Lower Seat Edge	450mm
Width Between Armrests	450mm
Cut-out Width in Seat Plate	215mm
Cut-out Depth in Seat Plate	255mm
Castor Diameter	125mm
<b>Maximum User Weight</b>	<b>190kg (30st)</b>

### Use of the Product

This product has been designed to provide an aid for toileting or for transfer indoors on even surfaces.

### Assembly Instructions

The product is delivered as the main frame with castors attached, a seat back and arm assembly, the footrests and the potty, see detail below.

### Adjusting the Product

Castors:

- The rear castors can be braked, using your foot to push down on the castor's lever, this will prevent the wheel from rotating and swivelling
- To release the brake the lever should



Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park  
Cowbridge, Vale of Glamorgan, CF71 7PF

E: [customerservices@integrity-healthcare.co.uk](mailto:customerservices@integrity-healthcare.co.uk)

T:+44(0)800 043 0050

be pushed upwards, again using your foot

- Brakes should always be applied with the castors facing forwards when entering or leaving the chair
- With the brakes applied the Glideabout may still slide on a smooth floor or if it tipped forward by the user. Supervision is essential when transferring in and out of the commode

#### Armrests:

- The armrests slot into the brackets mounted either side of the commode
- To allow side access / egress pull the spring clip knob backwards and allow the armrest to drop downwards
- When refitting an armrest pull the armrest upwards, ensure that the spring clip is locked in place and the armrest secure prior to further use



#### Footrests:

- To adjust the height of footrests, first loosen bolt at the base of the footrest until the footrest can move smoothly, slide the footplate to the required height and retighten bolt until the footrest is rigid
- For access footrests can be swung away from front of the Glideabout by using the catches at top of the footrest assembly. Footrest can also be removed completely by lifting the complete assembly



the  
the  
the  
the



#### Emptying the Waste Pot:

- The potty can be removed by sliding the whole potty to the rear of the Glideabout and removed for emptying and cleaning

#### Glideabout Height Adjustment:

- The entire unit may be raised (or lowered) by removing the bolt above the castors and then lengthening or shortening the castor stem. Align the holes on the castor extension with the lower tubes of the Glideabout frame and then replace the bolt.



- Ensure that all castors are at the same height.

#### Cleaning

To clean the product, please use a mild detergent with warm water and a soft cloth to wipe down the framework on at least a weekly basis or more frequently if needed. If required, disinfect with a chloride disinfectant, ensuring thorough rinsing and drying prior to reuse. Avoid using abrasives as these may damage the product.

#### Maintenance

This product is maintenance free if the cleaning and safety instructions shown above are followed. Check the product regularly for damage and to ensure that it is securely assembled. Check the brake function regularly to ensure castors lock securely & release easily.

**Thank you for choosing an Integrity product.**

EUCEREP B.V.  
Roald Dahllaan 33 , 5629MC  
Eindhoven  
Netherlands

UK | CE  
CA

UK | UK  
CA | NI

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park  
Cowbridge, Vale of Glamorgan, CF71 7PF

E: [customerservices@integrity-healthcare.co.uk](mailto:customerservices@integrity-healthcare.co.uk)

T:+44(0)800 043 0050