

Height and Width Adj. Toilet Frame with Injection Moulded Seat - 90342

Assembly Instructions and User Guide

Thank you for choosing this Integrity product. The user guide should be read carefully before use as it contains important information about the safe use and maintenance of the product.



Product Dimensions	
Overall Width	550mm
Overall Depth	520mm – 580mm
Seat Depth	440mm
Seat Height	410mm – 560mm
Width Between Arms	490mm
Footprint Max	580mm x 595mm
Maximum User Weight	190kg (30st)

Use of the Product

This toilet frame supports users to give them confidence as they sit and rise, offering support and a lessening of strain on limbs.

Assembly Instructions

1. Carefully unpack and examine the product to ensure it is complete.
2. Before use, check for any signs of damage e.g. cracked weld or flaking paint. Do not attempt to use if the product shows any sign of damage.
3. Remove the E-clips from the legs and select the desired height of the frame by gently sliding out each extension leg from the frame.
4. When the desired height is obtained, refit the E-clips into the appropriate hole ensuring that the pin clips go through each leg and its extension leg and that all legs are set to the same height.
5. The frame is now ready for use.

Cleaning

Use a mild detergent with warm water to wipe down the framework on at least a weekly basis or more frequently if required. Ensure the toilet frame is allowed to fully dry before use. The moulded seat can be lifted or removed for cleaning:



Thank you for choosing an Integrity product.

EUCEREP B.V.
Roald Dahllaan 33 , 5629MC
Eindhoven
Netherlands

UK | CE
CA

UK | UK
CA | NI

Integrity Mobility Solutions Ltd, Unit 47 Vale Business Park
Cowbridge, Vale of Glamorgan, CF71 7PF
E: customerservices@integrity-healthcare.co.uk
T:+44(0)800 043 0050